Multi Tray

37269/5014

Gebrauchsinformation

Directions for use

Mode d'emploi

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Deutsch2

English.....3


Français4

Italiano5

Español.....6

Português7

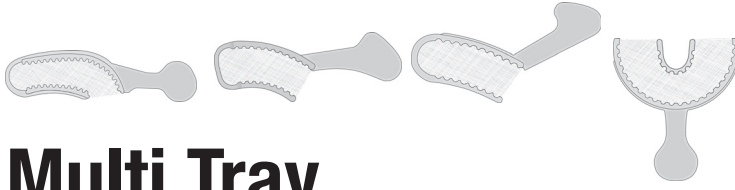Türkçe.....8

Multi Tray

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Multi Tray

Partial single-use tray for multifunctional application - impressions, opposing jaw impressions and bite registrations.

Indications/Techniques

- Impressions for fabricating **temporary** crowns and fixed partial dentures
- Partial anatomical impressions
- One-step precision impressions for fabricating fixed restorations (1-2 unit impression)

Instructions for use in dentistry

- ① Multi Trays are suitable for use with anatomical impression materials, e.g. Silginat® or also medium- to high-viscosity precision impression materials including in combination with low viscosity materials.
- ② When using a Multi Tray, check that the distal edge of the tray is lined up to the distal of the last upper tooth.
- ③ Instruct patient to bite on the unloaded impression tray and ensure that the occlusion is correct.
- ④ Load both sides of the tray adequately with the impression material (Figure 1).
- ⑤ Place the loaded tray into the patient's mouth within the working time of the impression material.
Ensure that the distal edge of the tray is in the correct position (Figure 2).
- ⑥ Ask the patient to bite firmly in the correct occlusion until the material has set fully (Figure 2).
- ⑦ After the material has set, carefully remove the impression tray from the patient's mouth (Figure 3).
- ⑧ Disinfect the impression.
- ⑨ Store the impression tray until fabrication of the prosthetic restoration is complete.

Warning

Multi Trays should not be reused. Following intraoral try-in, the impression tray must be used only for the respective patient or disposed of for hygienic reasons.

Multi Trays are unsuitable for use with putty materials.

Cautions

Do not use after the expiration date.

Multi Trays must be used with firm setting impression materials.

This product does not normally cause an allergic reaction; an allergic reaction to this product cannot, however, be ruled out with sensitive patients.

Storage conditions

Store dry and protect against sunlight. Store at room temperature. Avoid extreme temperature ranges.

Disinfection

Trays with impressions can be disinfected using for example, a 2% acidic glutaraldehyde solution or with a disinfectant solution intended specifically for impression materials (e.g., Silosept®) See manufacturer instructions.

For professional use only. Caution: Federal (U.S.A.)
 Law restricts this device to sale by or on the order of a dentist.

Made in Korea
 37269/5014

Trademarks




Silosept® and Silginat® are registered trademarks of Kettenbach GmbH and Co KG.

Note

Selected Kettenbach materials are available in certain markets only.

Legend

Conformity with MDD 93/42/EEC	CE	Use-by date	
Temperature limit		Keep dry	
Keep away from sunlight		Catalogue number	REF
Batch code	LOT	Consult instructions for use	
Dual system (Germany only)		Rx only For professional use only. Caution: Federal (U.S.A.) Law restricts this device to sale by or on the order of a dentist.	
Quadrant REF 17753	30 x	Posterior REF 17750	50 x
Sideless Posterior REF 17751	50 x	Anterior REF 17752	30 x
Do not re-use			

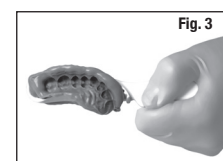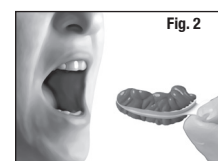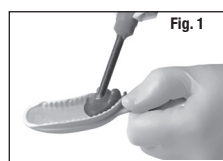Limitation of liability

Except where prohibited by law, Kettenbach GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence, or strict liability.

The information provided for Kettenbach products is based on comprehensive research and experience in application technology.

Results are furnished to the best of our knowledge, subject to technical changes within the framework of product development.

However, users must comply with and consider all recommendations and information in connection with any use.

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Silosept® ve Silginat®, Kettenbach GmbH und Co KG şirketinin tescilli ticari markalarıdır.