SAFE-ST SURFACE DISINFECTION WIPES NON-ALCOHOLIC

• Surface Disinfection Wipes non-alcoholic

- *Alcoholfree wipes for rapid disinfection of large surfaces*
- *Fast acting*
- *High degree of material compatibility*
- *Easy and practical to apply*

PRODUCT DESCRIPTION:

Alcohol-free disinfection wipes for rapid disinfection and cleaning of medical inventory and all kinds of surfaces in medical areas. Based on quaternary ammonium compounds Safe-ST Surface Disinfection Wipes non-alcoholic are free from alcohol and do not contain aldehydes or phenols. Due to the high-quality lint-free wipes, it guarantees release of active ingredients and excellent cleaning power, suitable for all professional applications.

AREAS OF APPLICATION:

Acc. to Medical Device Directive (MDD): Disinfection and cleaning of alcohol-sensitive, non-invasive medical devices. Acc. to Biocidal Products Regulation (BPR): For disinfecting and cleaning all types of alcohol-sensitive surfaces.

SPECTRUM OF APPLICATION AND EXPOSURE TIME:

Surface disinfection (wipe disinfection) bactericidal, yeasticidal per VAH [German Association for Applied Hygiene]: 1 min., effective against all enveloped viruses per the Robert Koch Institut recommendation incl. HBV/HIV/HCV/Vaccinia/BVDV/Influenza/flu viruses: 30 secs.

APPLICATION:

Thoroughly dampen surfaces and objects. After the application time has elapsed, wipe down with a disposable wipe, where necessary.

COMPOSITION:

100 g ready to use solution contains: 0.5 g Alkyl (C12-16) dimethylbenzyl ammonium chloride; 0.25 g Didecylmethylammoniumchloride.

SPECIAL INFORMATION:

Open the refill package. Shape the open package into a roll. Place the contents, including bag, into the dispenser. Pull the first cloth from the middle of the roll into the lid of the dispenser and replace the lid on the container. Please refill the container only with the refill package of Rapid Disinfection Wipes AF. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/ container to approved disposal company or local collection.

PRODUCT STATUS/LISTINGS:

Conform to the EU Biocidal Product Regulation (BPR, 528/2012).

Conform to the EU guideline 93/42/EC for medical devices.

AVAILABLE DELIVERY FORMS:

Itemnumber	Packaging
9797720	100 pcs refill pack
9797724	100 pcs container

Use disinfectants safely. Always read the label and product information before use.

SAFE-ST SURFACE DISINFECTION WIPES NON-ALCOHOLIC

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